



AED Sentinel[®]

Remote AED Monitoring System

from Readiness Systems

Zoll AED Plus Setup Guide

AT&T Project



info@readsys.com | readsys.com | (855)291-9100

What's in the Box




Zoll AED



CPR-D-padz –
aka “Electrodes”



10 Duracell CR123
Batteries

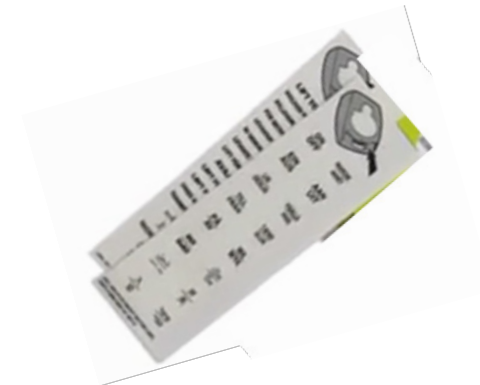

AED Plus ON/OFF
& LIFT TO OPEN
Label Sets

Apply Labels

Apply the “Lift to Open” and
“ON/OFF” labels as shown



Lift to Open the Lid



Remove Protective Film from Front of AED



Place Electrodes on Top of AED



Plug Electrodes into AED



Insert Electrode Expiration Tab Through Slot Place Lid Back on AED



Use Coin or Flathead Screwdriver to Remove Battery Door



Insert 10 Duracell CR 123 Batteries



Once all 10 batteries are placed,
the AED will perform an
audible self-test

IMPORTANT NOTE
Once the self-test is
complete,
**PRESS THE
BLUE BUTTON**



Replace Battery Door – AED Now Ready for Use



Confirm AED Readiness

Turn the AED over and confirm there is a green check mark in the status indicator window

